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Boite O-Ring : Polystyrène

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Common features

Clinically tested, these medical devices allow:

- apply a gentle, continuous force to correct the position of teeth,
- close or open spaces between teeth,
- ensure effective resistance to elongation.

Indications

- Latex-free
- Delivered clean, can be used directly without cleaning, disinfection or sterilization,
- If disinfecting, do not soak the product for more than one hour in a disinfectant solution.

Contraindications

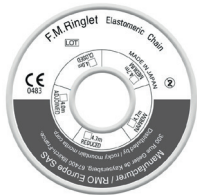
Contraindicated for maintaining a constant initial force.

Precautions for use

- Do not expose to direct light, sunlight or ultraviolet rays,
- Do not store in a very humid place,
- Store at room temperature,
- Do not reuse,
- Avoid using alcohol-based oral rinses, but use water instead.

Energy Chain™ F.M. Ringlet

- Rounded shape of the chainlet for interconnections,
- Variation of the space between the rings: allows an appropriate selection according to their application (ligation, retraction,...).


	Closed	Adjoined	Reduced	Medium	Large
Rings size	Small	Small	Small	Small	Large
Space between rings	Closed	Adjoined	1 mm	1,2 mm	1,2 mm
Space between centers	2,8 mm	3 mm	3,8 mm	4 mm	5,1 mm
Spool length	4 m	4 m	4,70 m	4,70 m	4,70 m

Tie-Thread

- Solid, round or square cross-section surface wire for better knot retention and use in rotations and retractions,
- Resistant to oxidation, odor and moisture.

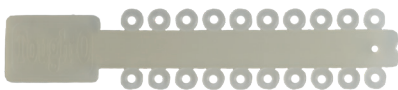Tough-O® Ligatures

- Greater strength than conventional ligatures,
- Diameter: .118" (3.0mm) diameter.

Elastomeric O-Ring

- Long lasting rings,
- Elastomer identical to Energy Chain™.

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