rocky mountain orthodontics  
**RMO**<sup>®</sup>

**Band Adapter**  
 Instructions for use  
 Las instrucciones  
 Anweisungen  
 Le istruzioni

**RMO, Inc.**  
 P.O. Box 17085  
 Denver, Colorado 80217-0085  
 USA

European Representative:  
**RMO Europe**  
 Rue Geiler de Kaisersberg  
 67400 Illkirch  
 France

ISN 263-00384 Rev. C PTD in USA

**ASENTADOR BANDA/ADAPTADOR**

Estimado Doctor (a):

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### **BAND SEATER / ADAPTER**

Dear Doctor:

RMO has modified the Band Adapter tip to allow you to replace the plastic tip at chair side. The replacement tip is RMO stock no. i00065.

A few considerations on installation and sterilization:

#### **INSTALLATION:**

Remove old tip. Cut if necessary. Start new tip over end of instrument with serrated surface opposite the pin. Push tip to allow slit to open and slide around pin.

#### **STERILIZATION:**

The i00064 instrument must be cold sterilized only. Remove tip. Sterilize all parts following instructions provided with the cold sterilization agent. It is important to thoroughly dry the tip and instrument before reinstalling a plastic tip.

#### **INSTRUCTIONS FOR STERILIZATION:**

Cold sterilization methods are recommended for the instrument body, as autoclaving temperatures are deleterious to the plastic materials. Please note—the tip section is made from a special hard, heat-treated stainless alloy. Even though stainless, moisture in the assembled threaded portion will cause a slow oxidation and result in seizure of the threads. The tip may be sterilized using cold or autoclave methods, but must be thoroughly dried before reassembly. A light application of silicone lubricant on the threads will help promote smooth operation and long service.

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