Nettoyage

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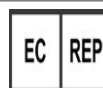
ContacEZ, LLC
217 SE 136th Ave, Suite 105
Vancouver, WA 98684
United States

Cleaning

Option 1. Clean ContactEZ *Dental Strips* in an Automatic Washer.

Option 2. Clean ContactEZ *Dental Strips* by hand.

Step 1. Holding the ContactEZ *Dental Strips* under a faucet, run regular tap water over each strip, and scrub the stainless steel surface using a brush.

Step 2. Clean the ContactEZ *Dental Strips* using General Cleaning Solution in an Ultrasonic Tank for 10 - 20 minutes.

Step 3. Rinse the ContactEZ *Dental Strips* under a faucet with regular running tap water.

Additional mention : refer to the next page.

Sterilization

Step 1. Sterilize the ContactEZ *Dental Strips* in a standard steam heat autoclave (not dry heat) at 30 psi max for 7 minutes, with a maximum temperature of 273°F/134°C.
ContactEZ recommends using distilled water in the autoclave.

Step 2. Allow the ContactEZ *Dental Strips* to cool down for a period of 5 minutes once the sterilization process is complete.

Step 3. Remove the ContactEZ *Dental Strips* from the autoclave by gloved hand. The strips are now ready to use.

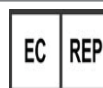
Disposal

When the diamond powder has worn off of the stainless steel strip, ContactEZ *Dental Strips* may be disposed of with normal waste products.

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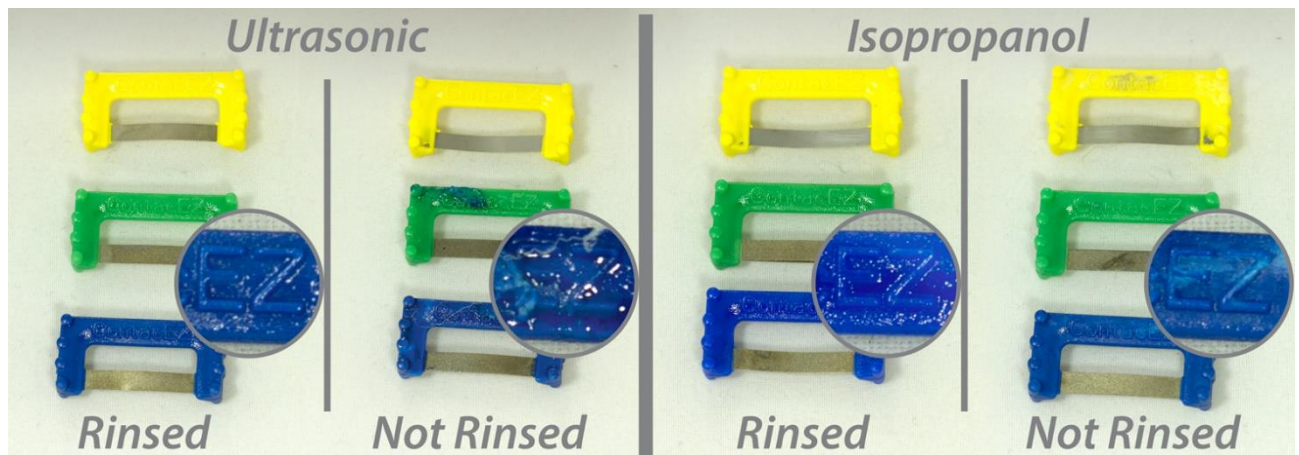
Exemples de dommage :

ADDITIONAL INFORMATION

MANUAL CLEANING FOR STRIPS BEFORE STERILIZATION

The strips should not be cleaned with a chemical disinfectant solution before sterilization.

During sterilization, residual chemical disinfectant including solution as chlorine bleach can damage plastic bond occurring deformation and/or breakage.

In case of using disinfectant : Rinse thoroughly the strips with water

Examples of damage :

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ContactEZ, LLC