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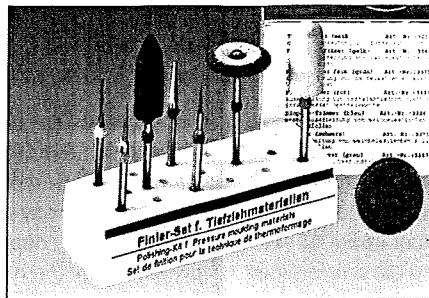
Finishing tools for pressure moulding materials

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For cutting and finishing of all pressure moulding materials, especially of splints and retainers made of IMPRELON® »S« as well as of foils bonded with IMPRELON® »S« (e. g. OSAMU-RETAINER®) we recommend the finishing set with 7 burs and stones marked in colours. The coloured marking enables easy choice corresponding to each process.

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1 OSAMU-Polierer (grau)

REF 3247

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Finishing-Set for pressure moulding materials

REF 3378

Content:

1 Cutting bur (white)

REF 3214

Rough cutting of hard foils

1 HM-carbide cutter (yellow)

REF 3369

Rough cutting of hard foils and hard / soft foils

1 HM-Tricutter (green)

REF 3370

Finishing of hard foils and splints

1 Finishing bur (red)

REF 3377

Cutting of hard foils and splints with integrated wire elements.

1 Bioperm-Trimmer (blue)

REF 3226

Rough trimming of soft foils and sandwich foils

1 Polyfix (black)

REF 3371


Finishing of soft foils and sandwich foils

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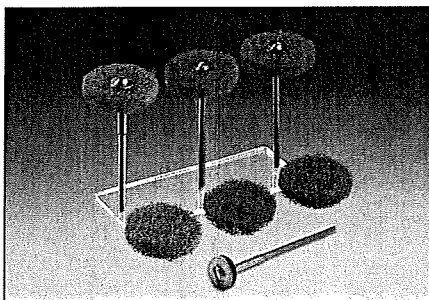DIMO® – these are the perfect abrasive wheels for professional finishing of pressure moulding foils, hard and soft plastic materials and metals. The soft elastic wheel with the open nonwoven structure are best suited for grinding and finishing of silicon, polyurethane and other soft remaining resins. The optimized material, the minimized unravelling as well as the low thermal development guarantee a much longer life of the wheels.

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
DIMO®PRO SLIM

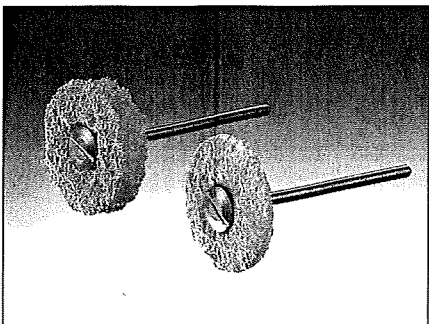REF 3376

The working time of DIMO®PRO wheels is many times longer than the one of comparable products. It's the combination of a special latex mixture and the fine nonwoven structure that makes the DIMO®PRO so versatile. Depending on the rpm, perfect grinding, smoothing and polishing is possible – using only one wheel: A real plus for the user.

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IMPRELON® »S« in the a. m. thicknesses is a high quality polycarbonate which is a break- and abrasionresistant clear thermoplastic. The material is best suited for all splint therapies.

Pre-drying: Due to the properties of polycarbonate and the high pressurizing temperature of approximately 180 °C **pre-drying** of IMPRELON® »S« is absolutely **necessary** to **avoid air bubbles** when plastifying this material. IMPRELON® »S« can be stored in the BIOSTAR-drying oven or, depending on the thickness of the material can be **pre-dried** for the recommended conditioning time in the BIOSTAR drying oven or a similar oven at a temperature of **80 °C – 100 °C**. It may be necessary to prolong first pre-drying time of 2.0 and 3.0 mm sheets depending on humidity, storing and transport conditions. Sheets prepared in this way can be used for approximately 7 days without any air bubbles. Repeated pre-drying is possible without any loss of quality.

Bite splints: The etching liquid PERLIBOND can be applied to IMPRELON® »S« if resin should be added to bite splint. For this use 1 – 2 times PERLIBOND for the occlusal areas and model the desired shape by applying autopolymerisate (e. g. DURASPLINT®, STEADY-RESIN) on the moistened bonding agent.

Polymerization and polishing is done as usual.

IMPRELON® »S« – as well as all our pressure moulding foils – has been tested at the University of Heidelberg with regard to toxic properties, gingiva irritations and mutagenity. The results clearly state that these foils are biologically well-accepted.

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REF 3236
REF 3237
REF 3238

For cleaning of splints made of IMPRELON® »S« we recommend our CETRON®-cleansing and care products.

CETRON® Cleansing set
CETRON® Cleansing powder
CETRON® Cleansing spray

REF 3235
REF 3239
REF 3238