
MASQUE ORTHOPÉDIQUE FACIAL[®]

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F.D.A. n° 8 020 822

C.E. n° B.I. 05861-97

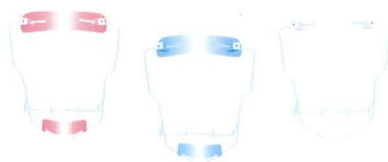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ORTHOPÉDIC FACE MASK®

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F.D.A. n° 8 020 822
C.E. n° B.I. 05861-97

“Extra-oral Orthodontic Headgear”®
is manufactured by France Orthodontie
RCS Marseille B 312 516 317 - NAF : 514 S
france.orthodontie@wanadoo.fr

This mask (the original one) features its simplicity and sturdiness and the possibility to be adapted to almost any face. It's the ideal choice for treatment of class III in children, before adolescence. Normal use is painless, without any risk (wound). Several thousand of treated cases bare witness of its innocuousness.

Appliance is designed for a single use, its re-use is not advisable.

Two models exist : standard and long faces.

A few precautions must be observed to obtain the best possible service from this instrument.

① ADJUSTMENT OF THE INSTRUMENT TO ADAPT IT TO CERTAIN PARTICULAR CASES

Height : the sides of the metal frame are provided with compensation curves. To lengthen the frame, the curve of both sides must be opened more or less with strong pliers. Converseley, the height can be reduced by accentuating the curve. This adjustment must be symmetrical.

Width : the width can be adjusted by sliding the desired amount of the metal rods engaged in the plastic pads including the brass inserts and the socket head screws, adjusted with the Allen wrench delivered with the mask.

After adjustment, the rods are secured by moderately tightening the screws in the inserts, stopping as soon as contact of the screw on the rod is felt. The end of the screw is provided with a bead which grips the metal of the rod to secure it.

Adaptation to the convexity of face : use a strong “three-jaw” pliers to slightly curve the side rods for perfect application of forehead and chin cup on the teguments.

The “three-jaw” pliers must be put approximatively 1/4 inch from the edges of forehead and chin cup.

② EXCESSIVE PERSPIRATION

The plastic used for the forehead and chin cup is of the food quality type, non-allergenic, perfectly safe.

However, certain skins may require protection against the effects of excessive perspiration which would lead to a certain maceration of the cutaneous area covered. To avoid a negative effect of perspiration, it is necessary to line the insert surfaces of the support with leather pads (delivered with each mask).

③ CARE AND CLEANING

To clean the mask, use sudsy water exclusively to the exclusion of any another product (particularly alcohol).

The elastic bands must be replaced daily.

If these very simple rules are complied with, the mask will give you full satisfaction.

Caution : “Federal law restricts this device to sale by or on order of a dentist”.