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# THE RAINMAKER – INSTRUCTIONS OF USE

## CHARACTERISTICS

**Single Use :** This device and its components are intended to be for a single use, because they are adapted in a custom made product for a specific patient.

The risk of cross-contamination exists. Sterilization / decontamination according to the regulations are recommended.


The «RAINMAKER» is an orthodontic medical device for mandibular advancement to be used in combination with individual aligners.

It has been designed in a spirit of comfort for the patient but also for its simplicity of assembly.

There is only one tube length that will need to be adjusted for a particular patient.

## COMPOSITION

### Each kit contains

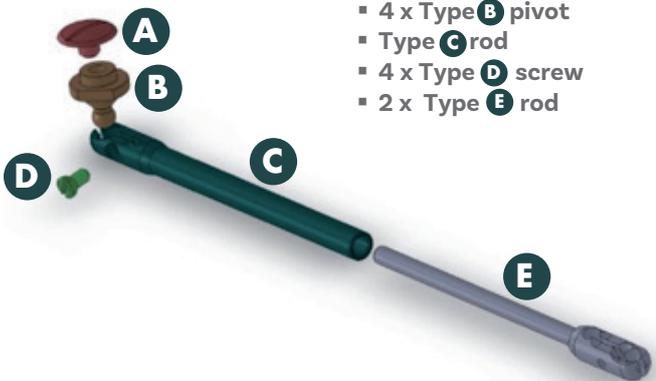- 4 x Type **A** screw
- 4 x Type **B** pivot
- Type **C** rod
- 4 x Type **D** screw
- 2 x Type **E** rod

Symbols	Description
	Not re-usable-single use
	Read the Instructions
	Electronic Instruction Indicator
	Mentions the catalogue part number
	Indicates the Lot Number
	Indicates presence of Nickel
	For the usage of only a qualified person
	Indicates the manufacturer of the Medical Device
	Indicates the number of the Certified Body

## INSTRUCTIONS

- Each aligner (Maxillary and Mandibular) must have two pivots. 2 pivots at the level of the molars on the maxillary, 2 pivots at the level of the canines on the mandible.
- Each pivot is attached to the aligner by the screw **A** on both sides of the aligner.
- Each aligner must have holes (x2) for the screws and a withdrawal zone in its thickness. The positioning of the screw **A** in this withdrawal zone allows the top of the screw to be in contact with the surface of the tooth.
- The rods are positioned on the ball joints of each pivot. Tightening the screw **C** up to the contact of the two branches makes it possible to secure the two parts. Check the tightness of each connecting rod on each pivot. They must be held well together.
- This operation is reproduced on all 4 pivots.
- A flat screwdriver covering the full width of the screw is recommended.
- During the transition to the next aligner, the 4 screws **D** are loosened and then tightened on the pivots of the aligner of next stage.
- Additional pivots can be ordered to equip in advance more aligners.
- This mandibular advancement system is generally used only for part of the treatment, according to the indications of the practitioner.
- At any time the parts can be disassembled and cleaned with the help of a toothbrush.
- **Possible problems:** In case of separation of parts, bring everything back to your practitioner who can check the assembly. In case of insufficient mandibular advancement determined by the practitioner, a wedge of different dimensions may be added.

Notice\_Rainmaker\_05/12/2018

B.P. 20334 - 300, rue Geiler de Kaysersberg 67411 Illkirch Cedex - France  
 T +33 3 88 40 67 30 F +33 3 88 67 86 96 E info@rmoeurope.com  
[www.rmoeurope.com](http://www.rmoeurope.com)