

TRULOCK™

BONDING SYSTEM (LIQUID/GEL ETCHANT)



DESCRIPTION:

Trulock™ Etchant contains 37% phosphoric acid for use on both dentin and enamel. The viscosity of **Trulock™ Etchant** provides ideal control to prevent the material from flowing onto exposed surrounding surfaces. It rinses away clean without leaving any residue. **Trulock™ Etchant** is available in blue and green colors, making it easily distinguishable from the tooth structure.

INDICATIONS FOR USE:

To etch enamel and dentin for adhesive restorations.

CONTRAINDICATIONS:

Trulock™ Etchant is contraindicated for use on soft tissue including oral mucosa, skin, eyes and dental pulp tissue.

WARNINGS:

Trulock™ Etchant contains phosphoric acid which can cause burns of soft tissue. Avoid contact with oral soft tissues, eyes or skin. If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water. In case of contact with eyes, seek immediate medical attention.

Always replace original cap tightly after each use.

GENERAL PRECAUTIONS:

1. U.S. Federal law restricts this device to sale by or on the order of a dentist.
2. Review instructions and SDS prior to beginning treatment.
Use only as directed.

ADVERSE REACTION:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.


INSTRUCTIONS FOR USE:

1. Prepare the tooth.
2. Isolate and dry tooth thoroughly with oil-free compressed air.
3. Remove cap and place the applicator tip* on the **Trulock™ Etchant** syringe. Apply **Trulock™ Etchant** to prepared enamel for 15 seconds or if applied to dentin etch for 10 seconds. Lightly agitating the etchant during its application can enhance its action.
4. Rinse thoroughly with water.
5. If using a bonding adhesive, follow the manufacturer's instructions to determine if the surface of the tooth is to remain moist or thoroughly dry.

STORAGE:

Store material between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (22°C/72°F). Do not freeze. See outer packaging for expiration date.

* Applicator tips are disposable and intended for single patient use.

 RMO, Inc
650 West Colfax Avenue,
Denver, CO 80204 USA
www.rmortho.com

 EC REP
RMO Europe
300 Rue Geiler de Kaysersberg
67400 Illkirch, France